

DIFF-SAFE® Blood Dispenser

U.S. Pat. 5,344,666 and other foreign patents apply

Product Code: 125

BULK CASE

Description: The DIFF-SAFE blood dispenser is comprised of a base of plastic and a blunt ended stainless steel canula. The device is NOT sterile.

Indications: This device is intended to provide a safe means for puncturing a closed collection tube and obtaining a drop of blood without the need for removing the stopper. The device produces a uniform drop of blood suitable for making a smear. It is compatible with both VACUTAINER® and HEMOGARD® brand tubes.

Contraindications: This is a single use device and is NOT to be used with multiple specimens or to be cleaned and reused. NOT intended for any purpose other than indicated.

Disposal: The DIFF-SAFE may be discarded while still inserted in the tube or removed and discarded separately. In either case, dispose in a sharps medical waste container.



Instructions:

1. HOLD tube in upright position
2. INSERT DIFF-SAFE into stopper
3. INVERT and position DIFF-SAFE on target surface
4. Gently PRESS tube downwards until a drop forms at the tip of the DIFF-SAFE.
5. RELAX pressure the instant drop touches target.
6. For uniform drop size, slowly relax pressure to allow DIFF-SAFE to suck back excess specimen before removing device from target surface.

WARNING:

1. WEAR GLOVES.
2. DO NOT INSERT WHEN TUBE IS UPSIDE DOWN.
3. FOR TUBES PREVIOUSLY OPENED AND RECAPPED, MAKE SURE THE STOPPERS ARE FULLY SEATED ON THE RIM. FAILURE TO DO SO MAY CAUSE EXCESS SPECIMEN TO BE DISPENSED ONTO THE TARGET.

*VACUTAINER & HEMOGARD are registered trademarks of Becton Dickinson & Company

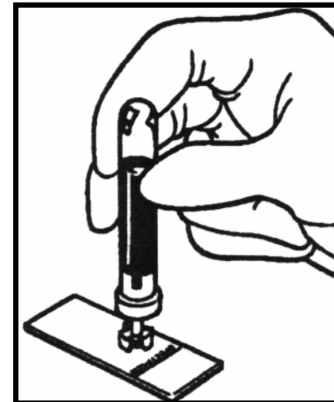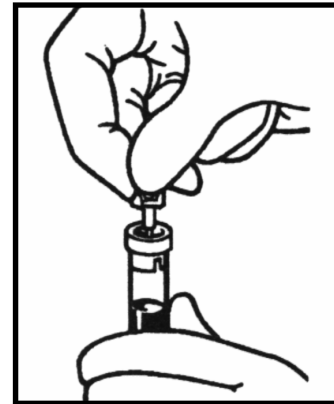

ALPHA SCIENTIFIC CORPORATION

P.O. Box 725, Southeastern, PA 19399-0725

Ph: 610-647-7000 Fax: 610-647-7496

Sales & Customer Service:

Ph: 800-242-5989 Fax: 800-841-2629

www.Alpha—Scientific.com

MADE IN USA



ASC 100021 - REV A 01/2004